

# Eye Drops



# Extra Moisturising

# For very dry & irritated eyes

Suitable for contact lens users

## **Eye Drops Extra Moisturising**

For relief of extremely dry, irritated or tired eyes. Provides quick, intensive moisturisation of the eye surface. Helps soothe and maintain lubrication of the eye. Suitable for contact lens users and those with sensitive eyes.

- 10 ml sterile solution
- No preservatives
- · Use within 6 months of opening

#### Extra refreshing and intensive moisturisation

Sometimes eyes can feel extra dry, irritated or tired, as a result of prolonged computer use, heating, air conditioning, age, wearing contact lenses, etc.

# With hyaluronic acid and Eyebright (Euphrasia)

A.Vogel Eye Drops Extra Moisturising with Eyebright (Euphrasia) and hyaluronic acid\*. The hyaluronic acid content is twice as high as in the regular A.Vogel Eye Drops, giving a more intensive effect.

\*The hyaluronic acid in A.Vogel Eye Drops Extra Moisturising is produced from a vegetable source. It does not contain qenetically modified soy, and is not of animal origin.

#### Unique dropper bottle

A.Vogel Eye Drops Extra Moisturising are packaged in a unique dropper bottle. The easy-to-use dropper system releases just one drop at a time for ease of application.

#### Use with contact lenses

If you wear contact lenses, you do not need to take them out before using these eye drops.

#### Dosage and use

**Adults:** 1 drop in each eye, up to 5 times daily. The drops can be used for a maximum of 30 consecutive days.

If your symptoms persist for more than 3 consecutive days without any improvement, or get worse, or if eyes become inflamed, you must contact your optician or healthcare professional.



1. Remove the coloured cap from the hottle



First time use: Hold the bottle upside down over your hand and squeeze at the base several times, until the first drop is released. The dropper bottle is now ready for use.



 Tilt the head back slightly and pull the lower eyelid downwards.
 Place the dropper above the eye.
 Holding the bottle with your thumb and middle finger, press on the base of the bottle with your index finger.



 Close your eye slowly so that the fluid spreads evenly over the eye's surface. Replace the cap on the bottle, pressing firmly.

What you need to know before you use
A.Vogel Eye Drops Extra Moisturising
Do not exceed the stated dose.

# Do not use if you are

Hypersensitive to any of the ingredients.

# Tolerability

The unique bottle design keeps the contents sterile without using preservatives, ensuring the product can be used frequently, once opened.

# Ingredients

Hyaluronic acid (2 mg/ml), *Euphrasia officinalis* (Eyebright), sodium chloride, boric acid, disodium tetraborate decahydrate, water.

No preservatives. Suitable for vegans.

## Storage

Keep out of the sight and reach of children. Store between 8°C and 25°C.

Use within 6 months of opening. Do not use after the expiry date.

# Packaging

A.Vogel Eye Drops Extra Moisturising is available in a 10 ml pack.

#### **Product Range**

A. Vogel Eye Drops are available as:

A.Vogel Eye Drops Extra Moisturising	For very dry and irritated eyes	10 ml
A.Vogel Eye Drops	For dry, irritated and tired eyes	10 ml

#### Personal advice

For further details or healthcare advice you can call the A.Vogel Helpline:

**UK:** 0845 608 5858

enquiries@avogel.co.uk **IE:** 1 890 930 070

enquiries@avogel.ie

Visit www.avogel.co.uk

### **Tips**

- Use natural light as much as possible. Do not work at a computer screen for long periods at a time. You can avoid this by taking regular breaks.
- Do not wear contact lenses for longer than the manufacturer's recommendation.
- Consider wearing sunglasses to avoid irritation from sunlight.
- Avoid air-conditioning where possible. Choose to open a window instead.
- · Don't watch too much TV.

#### Care for the environment

A.Vogel strives to minimise the impact on the environment. As far as possible our packaging materials are produced using environmentally friendly methods. These can be incinerated without any harmful impact on the environment or they are suitable for recycling.

The text of this leaflet was most recently revised on 08 May 2019.

\*\*\*

A.Vogel B.V.

J.P. Broekhovenstraat 16 8081 HC Elburg The Netherlands

#### Distributor:

**UK:** A.Vogel Ltd, Irvine, Ayrshire KA11 5DD **Ireland:** A.Vogel Ireland Ltd, Dublin 9

www.avogel.co.uk





STERILE A